

EI4001

FELIXCAN
ANIMAL ID

Automatic Implant Gun

DESCRIPTION

- Designed to be used in processes of microchips implantation when a great speed and a high reliability are necessary.
- It can be used with microchips of different sizes depending on their use.
- It is made of a high impact plastic. It gives it a great robustness inspite of being very light and it can be used in hard and aggressive environments.

HOW TO USE



Step 1.-

Unscrew the nut to free the clamp of the needle.



Step 2.-

Put the needle, with the screw-in looking at the interior of the implanter and push it softly until the end.

Before putting the new needle, check its point and blade. They must be in perfect conditions.



Step 3.-

Screw the nut to press the clamp of the needle.

It must be pressed with your hands until you realize that it is impossible to screw it more. NEVER use a tool to do this step because it could damage the needle and the clamp of the needle.



Step 4.-

Insert the cartridge on the top of the implanter.



Step 5.-

Slip down the cartridge with your finger until the upper part of the cartridge is at the same level as the upper part of the autoimplanter.

When you have implanted the microchip container in the cartridge, it will drop automatically and we should repeat the steps 4 and 5 to load it up again.

WARNING:

Before implanting a microchip, it is recommended to immerse the needle in water with a disinfectant solution.

Do not use the same needle for more than 25 implantations. Neither the point nor the bevel are prepared to last longer.

SPECIFICATIONS

Materials	High impact plastic.
Medidas	Size of the administrator without cartridge : 215x170x30mm Weight: 250gr Cartridge size: 2x12: 52x22x1mm Cartridge weight: Empty 4gr. Full 12gr

CERTIFICATES

- Felixcan boasts the ISO9002 quality certificates and AENOR IQNet, awarded for the "Design, Production and Marketing of Electronic Animal Identification Systems".
- Felixcan is granted the ICAR manufacturer code 941 and signed its code of conduct for the electronic animal identification.

CONTACT

- Polígono Industrial Romica, calle 1, Parcela 200, 02080 ALBACETE (España)
- Tlf.: +34 967 52 01 87 Fax: +34 967 52 20 61
www.felixcan.com admin@felixcan.com